LUKAS

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LX SWANGARD

de – Bedienungsanleitung | **en** – Operating instructions
sv – Bruksanvisning | **fi** – Käyttöohje | **el** – Οδηγίες χρήσης
hr – Upute za uporabu | **et** – Kasutusjuhend | **lv** – Lietošanas instrukcija
it – Istruzioni per l'uso | **nl** – Gebruiksaanwijzing | **da** – Betjeningsvejledning
fr – Manuel d'utilisation | **es** – Manual de instrucciones | **pt** – Manual de instruções
ro – Manual de utilizare | **bg** – Ръководство за обслужване | **sl** – Navodilo za uporabo
lt – Naudojimo instrukcija | **pl** – Instrukcja obsługi | **cs** – Návod k obsluze | **sk** – Návod na obsluhu
hu – Használati utasítás | **zh** – 原版操作说明书 | **ko** – 사용 지침 | **ja** – 取扱説明書 | **ar** – دليل الاستعمال

de
en
fr
es
pt
it
nl
da
sv
fi
el
pl
cs
sk
hu
ro
bg
sl
hr
et
lv
lt
zh
ko
ja
ar

Deutsch.....	4	de
English	12	en
Français.....	20	fr
Español.....	28	es
Português	36	pt
Italiano	44	it
Nederlands	52	nl
Dansk.....	60	da
Svenska.....	68	sv
Suomi.....	76	fi
Ελληνικά	84	el
Polski	92	pl
Čeština.....	100	cs
Slovensky	108	sk
Magyar.....	116	hu
Română	124	ro
Български	132	bg
Slovenščina	140	sl
Hrvatski.....	148	hr
Eesti.....	156	et
Latviski.....	164	lv
Lietuvių	172	lt
中文.....	180	zh
한국어.....	188	ko
日本語	196	ja
عربي	204	ar

INHALT

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CONTENTS

1.	Danger classifications	13
2.	Product safety	14
3.	Proper use.....	16
4.	Skills required by the user	16
5.	Use of the Swangard.....	16
6.	Instructions for using the Swangard with the "tunnel" extrication technique ...	17
7.	Care	19
8.	Inspection of the Swangard.....	19

1. DANGER CLASSIFICATIONS

We differentiate between various different categories of safety instructions. The table shown below provides an overview of the assignment of symbols (pictograms) and signal words to the specific danger and the possible consequences.

Pictogram	Damage/ injury to	Keyword	Definition	Consequences
	Persons	DANGER!	Immediate danger	Death or severe injury
		WARNING!	Potentially dangerous situation	Potential death or serious injury
		CAUTION!	Less dangerous situation	Minor or slight injury
	Property	ATTENTION!	Risk of damage to property/ environment	Damage to the device, damage to the environment, damage to surroundings
	-	NOTE	Handling tips and other important/ useful information and advice	No injury/damage to persons/ environment/ device

Wear a helmet with face protection

Wear protective gloves

Wear protective footwear

Proper recycling

Protect the environment

Read and follow the instruction manual

2. PRODUCT SAFETY

LUKAS products are developed and manufactured to ensure the best performance and quality when used as intended.

Operator safety is the most important consideration of the product's design. Furthermore, the instruction manual is intended to help you use LUKAS products safely.

In addition to the operating instructions, all generally applicable, legal and other binding regulations pertaining to the prevention of accidents and protection of the environment must be observed and operators must be instructed accordingly.

The SWANGARD rescue vest may only be used by persons who have been trained in the relevant safety regulations – otherwise, there is a risk of injury.

We advise all users to read the operating instructions carefully before using the product and to follow the instructions contained therein without any restrictions.

We also recommend that you have a qualified trainer teach you how to use the product.

WARNING / CAUTION!

The operating instructions for the other equipment and accessories used must also be observed.

Even if you have already received instructions on how to use the equipment, you should still read through the following safety instructions again.

	<p>Wear protective clothing, a safety helmet with visor, protective footwear and safety gloves.</p>	<p>Inspect the product before and after use for visible defects or damage.</p>	
 	<p>Immediately report any changes that occur (including changes in operating behavior) to the appropriate persons/departments! Decommission the product immediately if appropriate.</p>	<p>Do not carry out any changes (additions or conversions) to the product without obtaining the prior approval of LUKAS.</p>	
 	<p>Observe all safety and danger information on the product and in the operating instructions.</p>	<p>All safety and danger instructions on the product must always be complete and in a legible condition.</p>	
 	<p>Safety devices must never be disabled!</p>	<p>Refrain from any working methods that could impair or endanger the safety of the product!</p>	

	Make sure that the product will not put anyone in danger before and during use.	Only genuine LUKAS accessories and spare parts are to be used for repairs.	 
 	When working close to live components and cables, suitable measures must be taken to avoid current transfers or high-voltage flashovers to the product.	Observe all intervals for recurring tests and/or inspections that are prescribed or stated in the instruction manual.	
	Make sure that there is adequate lighting while working.	When working with and/or storing the product, ensure that the function and safety of the product are not impaired by the effects of severe external temperatures and that the product is not damaged in any way.	
	Always keep this instruction manual easily accessible at the place of use of the product.	Dispose of all parts as well as packaging materials properly.	 

In addition to the safety instructions in these operating instructions, all **generally applicable, legal and other binding national and international accident prevention regulations** must be observed and operators must be instructed accordingly.

WARNING / CAUTION / ATTENTION!

This product is intended **exclusively** for the purpose described **in the instruction manual (see chapter "Proper use")**. Any other use is **not in accordance with its proper use**. The manufacturer/supplier is not liable for any damage resulting from improper use. The user bears sole responsibility for such use.

Proper use includes observance of the instruction manual and compliance with the inspection and care conditions.

 **Never work in a fatigued or intoxicated state!** 

3. PROPER USE

This manual contains general information on the use, effect and care of the SWANGARD.

The manual shows the use of the SWANGARD in an ideal environment. Other, sometimes unusual circumstances may occur in practice. The SWANGARD can be used in many of these cases. It is the responsibility of qualified medical and fire department personnel to assess the condition of the patient and decide on the appropriate equipment and procedure.

The safe and proper use of the SWANGARD is at the sole discretion of the user. All other safety measures that the user takes must comply with the applicable regulations.

It is recommended that training in the correct use of the product is completed before using the SWANGARD in a real situation.

Keep the manual in a safe place for future reference. Include it with the product in the event of transfer to new users.

WARNING / CAUTION / ATTENTION!

Untrained users could be injured or suffer other damage to health.

Only allow trained personnel to use the SWANGARD.

Improper use of the SWANGARD can cause injuries.

Only use the SWANGARD for the purpose described in the manual.

4. SKILLS REQUIRED BY THE USER

- must have sufficient knowledge of procedures for the treatment of emergency patients (ABCDE procedures)
- must have the ability to help a patient

5. USE OF THE SWANGARD

- Carefully read the instructions
- Check the SWANGARD for defects (carry out an inspection in accordance with the instructions in this manual)

Train and practice using the SWANGARD rescue vest

ATTENTION!

Follow the guidelines of the medical director when using the SWANGARD. If recommended, manually stabilize the head and/or neck of the patient or apply a stiff neck collar while you put on and use the SWANGARD.

6. INSTRUCTIONS FOR USING THE SWANGARD WITH THE "TUNNEL" EXTRICATION TECHNIQUE

de

en

fr

es

pt

it

nl

da

sv

fi

el

pl

cs

sk

hu

ro

bg

sl

hr

et

lv

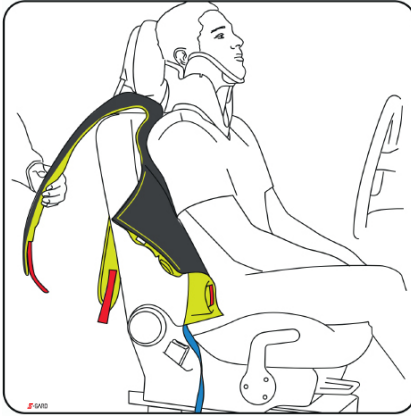
lt

zh

ko

ja

ar


STEP 1:

Open up the vest.

Place the vest between the seat back and the patient in line with the patient's shoulders. Hold the patient's upper body slightly forward while doing so.

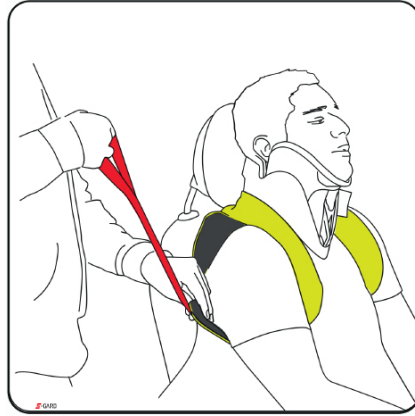
Use the blue tapes to pull the vest down to the sitting surface of the seat between the seat back and the patient.

STEP 2:

Use the red tapes to pass the shoulder straps under the armpits towards the back. Pull the red tapes out of the shoulder straps and allow them to hang down over the seat back.

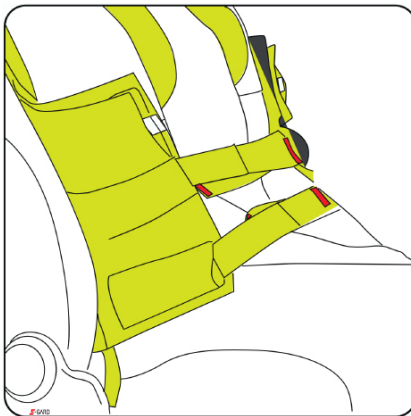Place the forearms on the patient's lap and wrap the sides of the vest around the patient in such a way that the arms are under the vest.


STEP 3:

Close the vest with the four Velcro® fasteners incorporated in the front of the vest.

ATTENTION!

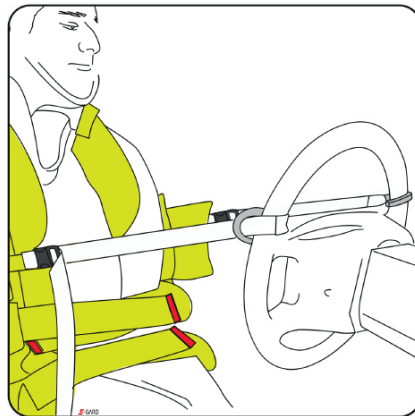Do not close the straps too tightly. Prevent breathing problems for the patient.


STEP 4:

Take the two straps with hooks from the pockets attached to the vest and hook them in.

The pockets can be removed, so that the straps can be attached to the vest at any desired angle.

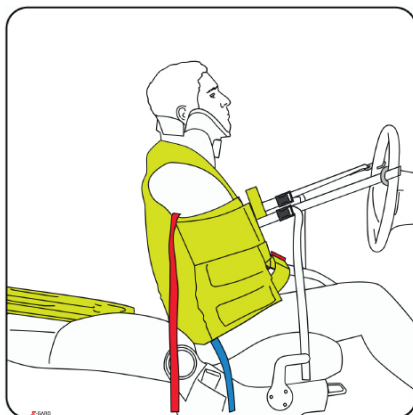
Now tighten the straps, so that the patient is pulled slightly off the seat back.

STEP 5:

Move the seat back into the reclined position or remove the seat back if this is no longer possible.

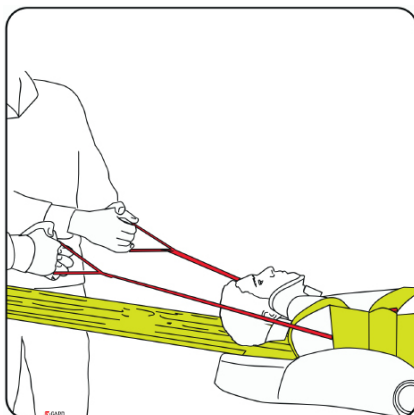
Place the spineboard in position, support the patient, release the straps by pressing the fasteners and lay the patient on the spineboard.

STEP 6:

Use the red straps to pull the patient on the spineboard.

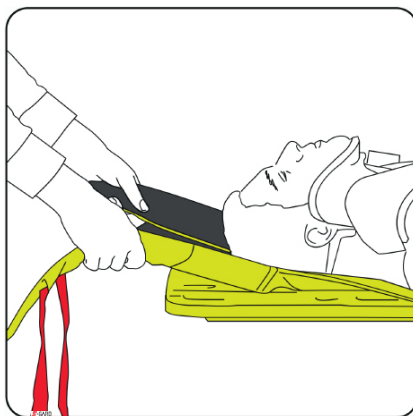Place the head end of the spineboard with the patient on the foot of a stretcher with a vacuum mattress and pull the patient over the spineboard onto the vacuum mattress.

STEP 7:

Open the vest, remove the shoulder straps under the armpits and use the red straps to pull the vest out from under the patient.

Note:

If it is not possible to extricate the patient with the tunnel technique, but the patient has to be extricated from the vehicle sideways, you can wrap the red tapes around their hands a few times, so that the pull lines are shorter. This provides a firm hold, in order to be able to remove the patient from the vehicle.

6.1 Putting the SWANGARD on a pregnant patient

Closure of the vest depends on the size of the abdomen and the chest of the patient. In most cases, all that is required is to close the straps in such a way that both arms are enclosed by the vest.

WARNING / CAUTION / ATTENTION!

An unprotected patient can be injured further.

Stay with the patient at all times.

Do not attach any devices or objects to the SWANGARD.

The attachment of other devices or components to the SWANGARD could cause injuries.

Please note that the SWANGARD does not fit every patient.

7. CARE

The SWANGARD must be washed after use.

Follow the supplied cleaning and disinfection instructions. Allow the SWANGARD to dry completely.

ATTENTION!

Storage of a damp or wet SWANGARD in the carrying case promotes the growth of mold. Do not put the SWANGARD back in the carrying case until the vest is completely dry.

7.1 Information on care

If you still have any questions about care or maintenance, please contact your authorized dealer or LUKAS Customer Service directly.

The address for the LUKAS Customer Service department is:

LUKAS Hydraulik GmbH

Weinstrasse 39, 91058 Erlangen, Germany

Tel.: (+49) 09131 / 698 - 348

Fax.: (+49) 09131 / 698 - 353

<http://www.lukas.com>

8. INSPECTION OF THE SWANGARD

- Are all the components present?
- Check that the buckles and hooks have no obvious damage and function as intended
- Is the vest free of excessive wear and are the seams intact?
- Is the vest clean?

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A Unit of IDEX Corporation